

Secure Denture Bonding Cream vitamins and Herbs.

[Log In](#) | [Log Out](#) | [My Account](#) | [Cart](#)

VITANET[®]
235 MARKET AVE. SW - HARTVILLE, OH 44632

100% Money
Back Guarantee
On All Products

LOCAL: 330-877-8786
FAX: 330-877-8787

1.800.877.8702
Free shipping on orders over \$100!!*

Secure Denture Cream

[Similar vitamins to Secure Denture Bonding Cream](#)
[General Information and datasheet \(BIOFORCE USA\)](#)

Check out **PayPal**
with

The safer, easier way to pay

Itemnumber: **BF0065**

Manufacturer: [BIOFORCE USA](#)

Description: **Secure Denture
Bonding Cream**

UPC: **047701000526**

Size: **1.4 oz**

Suggested Retail: **\$9.49**

Discount: **31% OFF MSRP**

Our Price: \$ 6.55

Quantity: **Add To Cart**

(5 for \$ 6.17ea.= \$ 30.85) 35%
OFF MSRP

Add To Cart



Ships within 48 - 72 Hours*

Quantity Available: 190

Other Popular vitamins Like Secure Denture Bonding Cream!

General Information and datasheet (BIOFORCE USA)

Great For Uppers, Designed For Lower

SECURE® creates a strong bond between the gum and denture which eliminates slipping and sliding for the main problem area, your lower dentures.

SECURE is the only denture adhesive that will not dissolve in water, so it will not wash out while eating or drinking.

SECURE is flavor free, so your mouth can feel the way nature intended.

Feel secure all day long!

Guarantee:

U.S. DenTek® is confident this product will perform to your satisfaction. We will refund your purchase price if you are not completely satisfied. Guarantee good only in United States. Send refund request and register receipt (within 30 days of purchase date) to: U.S. DenTek® Customer Service, 307 Excellence Way, Maryville, TN 37801.

INSTRUCTIONS:

Clean and dry your dentures thoroughly. Secure can only develop its full adhesive power when the dentures are completely dry.
Apply Secure to your dentures by squeezing the tube slightly. Warm the tube in hand if it is cold to let the adhesive come out easily.
Apply Secure to your dentures in strips (as shown in the diagrams) or series of dots, making sure that the adhesive is not too close to the edge of the denture.
To avoid clogging, close tube tightly immediately after use.
Insert dentures into mouth and bite firmly.


DataSheet:


The Secure Denture Bonding Cream Sale Price: \$6.55 - Vitamins or Herbs Should be taken as directed on the bottle.

[Vitanet Online.com](http://VitanetOnline.com), VitaNet ®, LLC

[Solaray](#) - [Ultimate Nutrition](#) - [Actipet Pet supplements](#) - [Action Labs](#) - [Sunny Greens](#) - [Thompson nutritional](#) - [Natural Sport](#) - [Veg Life Vegan Line](#) - [Premier One](#) - [NaturalMax](#) - [Kal](#)

[Support](#) -- Copyright © 1995-2008. VitaNet®, LLC. All rights reserved.


1 peter 1:15-16

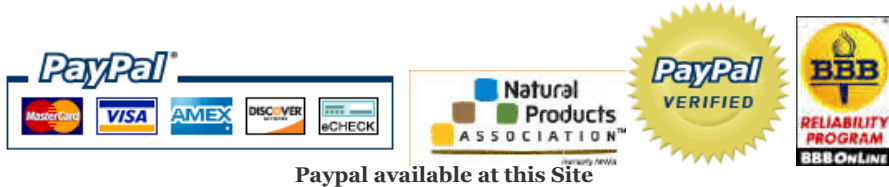

Paypal available at this Site

[VitaNet Is an Authorized Distributor of Secure Denture Bonding Cream](#) from [BIOFORCE USA](#) Products.

Disclaimer: Statements and vitamins contained herein have not been evaluated by the Food and Drug Administration. These discount vitamins are not intended to diagnose, treat and cure or prevent disease. Always consult with your professional health care provider before changing any medication or adding Vitamins to medications.